# PMMA DISC

## CE 0477

**88** by **8853**  
*dent* <sup>Spa</sup>

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GB - PMMA discs for the CAD-CAM processing of long-lasting temporary crowns and bridges made with a cross-linked high molecular weight resin that ensures a reliable reproduction of details and resistance even in the event of low thicknesses.

FIELD OF APPLICATION: Mainly used to manufacture more units (partial or complete) of long-lasting anatomical and temporary bridges. Class II A classification according to the Directive 93/42/EEC on medical devices and ISO 20795/2.

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### MAIN FEATURES

- Medical device intended for CAD / CAM dentistry: the dental laboratory handles the processing by means of a drill to achieve temporary and/or permanent teeth or molded structures, such as crowns and temporary bridges.
- High stability.
- Permanent colors.

### INSTRUCTIONS FOR USE

1. The material must be processed in a controlled environment to avoid contamination clean the material with alcohol if contaminated with oils and fats.
2. Sterilize the product before using it in the oral cavity.
3. The PMMA Discs can be sterilized with Ethylene Oxide or by means of ionizing radiation.
4. The product is not autoclavable.
5. Use brushes with goat hair to polish it and avoid excessive overheating.
6. Avoid cleaning with steam.
7. Any of the dental cements or adhesives available on the market may be used for the cementation of temporary elements. Use eugenol-free cements when performing a permanent dental cementation.

STORAGE: It is advisable to store the product at a temperature of 5 to 45 ° C. The PMMA Disc is not a dangerous product. It should be disposed of in accordance with the laws and regulations in force. It can be disposed of with domestic wastes or recycled as a normal plastic.

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